TUBULAR BANDAGE

MEDIPLAST

EN: Tubular bandage. **Intended purpose.** The Tubular elastic bandages are intended to be used for secondary fixation and as skin protection under cast and splints. (Danagrip: also to give light support to joints and ligaments)

To be used on intact skin. Cut the bandage to the length required for the area to be covered. Open and pull the bandage over the area. **Precautions:** The bandage should be kept clean. If contaminated with bodily fluids it should be discarded and replaced by a new bandage. In case of a serious incident connected to the use of the bandage, it should be reported to Medioplast AB. **Material** (see packaging).

DANAFAST DS Bamboo: Bamboo, Polyamide, Elastane. **DANAFAST DS Viscose:** Viscose, Polyamide, Elastane. **DANAGRIP Bamboo:** Bamboo, Polyamide, Elastane. **DANAGRIP Bleached Cotton:** Cotton, Polyamide, Elastane. **DANAGRIP Unbleached Cotton:** Cotton, Polyamide, Elastane. **DANATUBE Bamboo:** Bamboo. **DANATUBE Viscose:** Viscose. **DANATUBE Bleached Cotton:** Cotton. **DANATUBE Unbleached Cotton:** Cotton.

Waste Handling. The device shall be considered contaminated after use and shall be disposed of according to the procedure in place at the healthcare facility for handling contaminated waste.

Reporting. In case of a serious incident in relation to the device, report immediately to Medioplast and the local competent authority.

Duplicate copies of this instruction for use can be ordered free of charge.

This instruction for use has been prepared in English and subsequently translated into other languages. In the event of inconsistencies of interpretation, the English version shall be the definitive version.

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Australian Sponsor: Mediplast Australia Pty Ltd
Mediplast AB